

GAME CHANGER

Carriere® System Archwires with the Innovative EZ Stops™ System

The following archwires complement the *Carriere* philosophy of fast, gentle, and natural tooth movement.



- **Cu™ Nitanium® 27°C Archwires** - This wire series provides optimal nickel titanium forces with the additional flexibility required for easy placement in crowded cases, and comfortable tooth movement.
- **Cu Nitanium 35°C Archwires** - This wire series is activated at normal body temperature to facilitate easy engagement, consistent forces, and efficient tooth movement using transitional temperature changes back and forth between the "soft" martensite state and "firm" austenite state.
- **CNA® Beta III Archwires** - This wire displays superb formability for final adjustments. More flexible than stainless steel, but once intended shape is formed, the archwire maintains its integrity. The wire has a smooth, high-polished finish which provides less friction making it the perfect archwire for use with self-ligating brackets.



Easy positioning


- **Bio-Kinetix® Thermally Activated Nitanium Archwires** - An excellent finishing wire due to its greater flexibility and minimal stiffness. The cross section fills the bracket slot to fully express the final torque, and correct any rotational needs.

SLX Cu Nitanium Archwires are also available with our optimal EZ Stops delivery system which incorporates tube stops pre-loaded onto a disposable holder, packaged pre-threaded on each wire.




Twist to remove holder

- No more frustration with sticky sliding
- Easy to see
- Easy to grip
- Sterilizable by steam autoclave

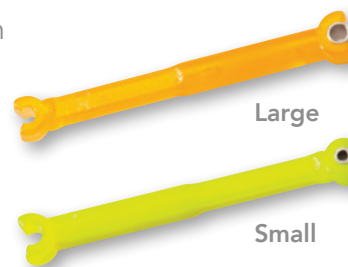

Crimp

Most Frequently Used SLX System Wire Sequence

Arch Form		 D-LX™ Shape Universal (Damon® Shape)
Packaged		10/Single Pack Pouches
Phase 1 (10-20 weeks)		Initiate tooth movement; begin aligning and leveling teeth; begin correcting rotations; start maxillary transverse arch development.
1st Wire	103-192C	.014 SLX Cu Nitanium 27°C with EZ Stops Archwires
Phase 2 (20-30 weeks)		Finish aligning/leveling; resolve remaining major rotational issues; begin torque control and space consolidation in the anterior; correct angulations; continue arch development.
2nd Wire	103-195C	.014 x .025 SLX Cu Nitanium 27°C with EZ Stops Archwires (Place in well-prepared arches.)
3rd Wire	103-200C	.017 x .025 SLX Cu Nitanium 35°C with EZ Stops Archwires (To express additional torque & prepare for CNA Wires.)
4th Wire	103-201C	.019 x .025 SLX Cu Nitanium 35°C with EZ Stops Archwires (To express additional torque & prepare for CNA Archwires.)
Phase 3 (10 weeks)		Begin major mechanics to correct A/P; Make most final torque adjustments; consolidate space closure in posterior; adjust buccal/lingual discrepancies; establish patient-specific arch form.
5th Wire	103-120C	.019 x .025 SLX CNA Beta III Archwires
Phase 4 (2 weeks)		Correct final minor rotations that are sometimes required when employing self-ligating bracket systems.
6th Wire	103-123C	.021 x .027 SLX Bio-Kinetix Thermally Activated Nitanium Archwires



SLX Archwires are great for use with the *Carriere Self-Ligating Bracket System*.



Subscribe to our eNews: HenryScheinOrtho.com/eNews

For more information on our products and educational offerings, please contact us:
HenryScheinOrtho.com
CarriereSystem.com
 Customer Service: 888.851.0533
 In the U.S.: 800.547.2000
 Outside the U.S.: +(1) 760 448 8600
 Canada: CERUM 800.661.9567

To email an order:
USASales@HenryScheinOrtho.com
 To email an order outside the U.S.:
IntlOrders@HenryScheinOrtho.com



HENRY SCHEIN®
 ORTHODONTICS

The New Movement In Orthodontics™