


Carriere® Motion™ Elastics Protocols

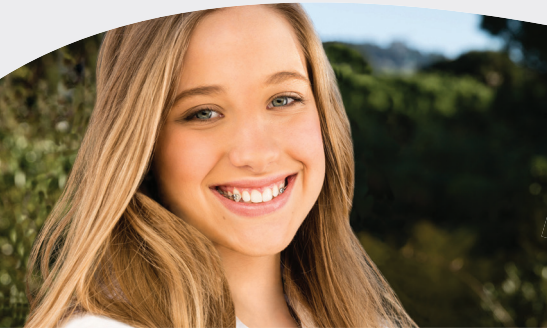

Motion Class III Appliance



Motion Class II Appliance



Motion CLEAR Class II Appliance


Elastics Protocol for Class II Cases

An elastic will run from upper arch to the lower 7. If the lower 7 is not available, run the elastic from the upper arch to the lower 6.

Standard Protocol


Motion Upper 3 to 6 with Tube on Lower 7

- 1st month: Force 1 elastics (6 oz, 1/4")
- After 1st month: Force 2 elastics (8 oz, 3/16") thereafter

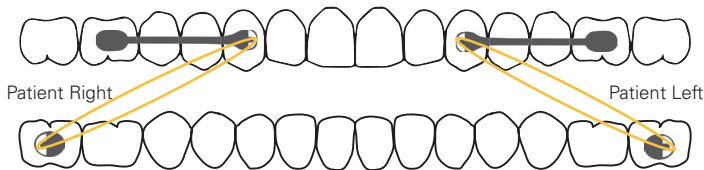

Motion Upper 3 to 6 with Tube on Lower 6

- 1st month: Force 1 elastics (6 oz, 1/4")
- After 1st month: Force 2 elastics (8 oz, 3/16") thereafter

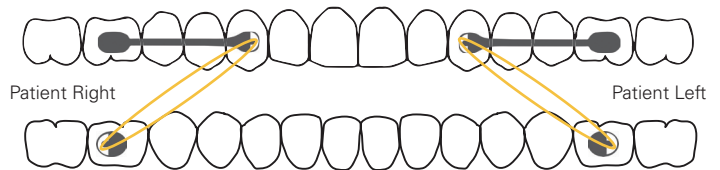


Blocked-Out Canine Protocol

Due to blocked-out, high, or buccally-displaced cuspids


Motion Upper 4 to 6 with Tube on Lower 7

- 1st month: Force 1 elastics (6 oz, 1/4")
- After 1st month: Force 2 elastics (8 oz, 3/16") thereafter

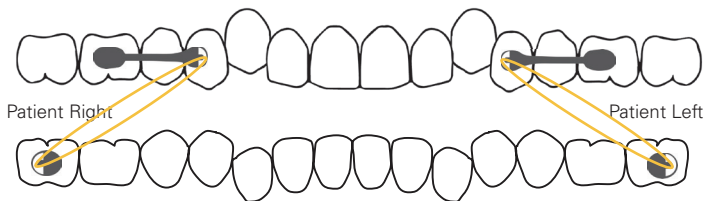

Motion Upper 4 to 6 with Tube on Lower 6

- 1st month and thereafter: Force 2 elastics (8 oz, 3/16")

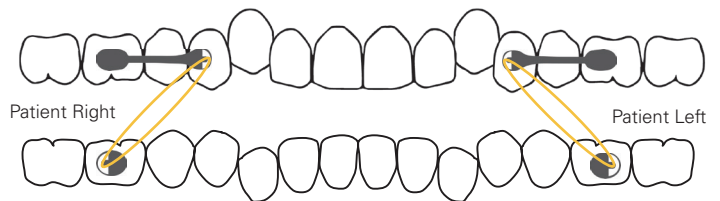


Mixed Dentition Protocol with Deciduous Canine

2/3 of deciduous canine's root must be available


Motion from Deciduous Canine with Tube on Lower 7

- Force 1 elastics (6 oz, 1/4") throughout the treatment

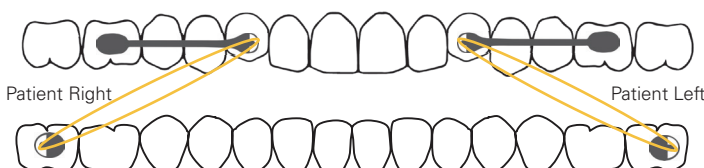

Motion from Deciduous Canine with Tube on Lower 6

- Force 1 elastics (6 oz, 1/4") throughout the treatment

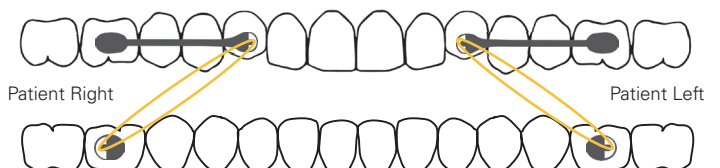


Adult Patients with High-Bone Density

If there is no movement after three months following the standard protocol in Class II, Division II, high-bone density patients, boost the case by proceeding with the following:


Motion 3 to 6 with Tube on Lower 7

- 4th month-night: double up Force 1 and Force 2 elastics (6 oz, 1/4" & 8 oz, 3/16")
- 4th month-day: single wear of Force 2 elastics (8 oz, 3/16")
- 5th month and thereafter: revert to single wear of Force 2 elastics (8 oz, 3/16")

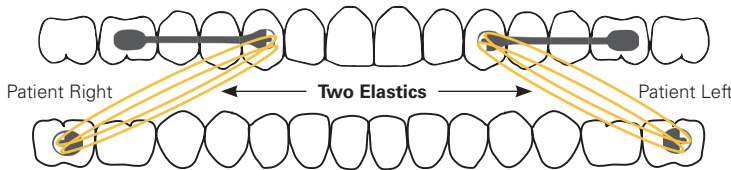

Motion 3 to 6 with Tube on Lower 6

- 4th month-night: double up Force 2 elastics (8 oz, 3/16")
- 4th month-day: single wear of Force 2 elastics (8 oz, 3/16")
- 5th month and thereafter: revert to single wear of Force 2 elastics (8 oz, 3/16")

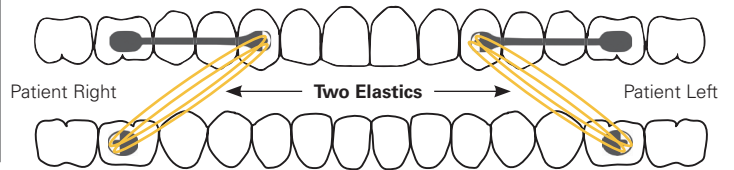


Elastics Protocol for Class III Cases

Elastic will run from the lower cuspid to upper molar

Standard Protocol


Motion Lower 3 to 6 with Tube on Upper 6

- Use Force 1 (6 oz, 1/4") elastics throughout the treatment

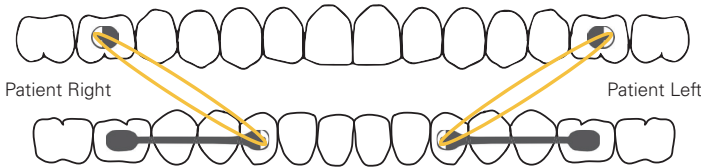

Motion Lower 3 to 6 with Tube on Upper 7

- Use Force 1 (6 oz, 1/4") elastics throughout the treatment

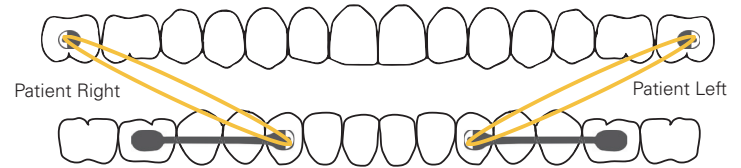


"Shorty" Cases Protocol

Elastic will run from 1st lower bicuspid to upper 1st or 2nd molar


Motion Lower 4 to 6 with Tube on Upper 6

- Use Force 2 (8 oz, 3/16") elastics throughout the treatment

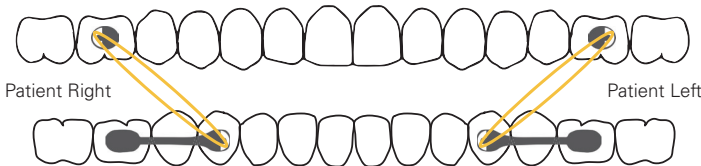

Motion Lower 4 to 6 with Tube on Upper 7

- Use Force 1 (6 oz, 1/4") elastics throughout the treatment

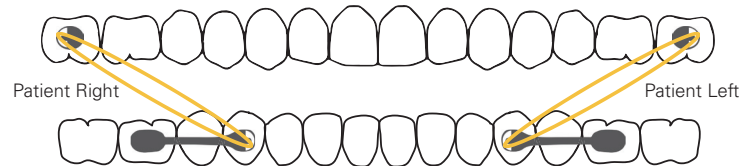

HENRY SCHEIN[®]
ORTHODONTICS

Rely on Us[™]

The New Movement In Orthodontics[™]

For more information on our products and educational offerings, please contact us:

HenryScheinOrtho.com | Customer Service: 888.851.0533

In the U.S.: 800.547.2000

Outside the U.S.: +(1) 760.448.8600

Canada: CERUM 800.661.9567

To email an order: USASales@HenryScheinOrtho.com

To email an order from outside the U.S.: IntlOrders@OrthoOrganizers.com

Ortho Organizers, Inc. 1822 Aston Ave. Carlsbad, CA 92008-7306 USA

© 2017 Ortho Organizers, Inc. All rights reserved. PN 999-325 Rev. A 2/17.

U.S. Patent No. 7,618,257, 6,976,839, and 7,238,022 and foreign patent numbers 2,547,433, 1723927, and 2006202089.

