  
Ortho Organizers, Inc.  
1822 Aston Avenue  
Carlsbad, CA 92008-7306  
USA  
Tel: 800.547.2000, +(1) 760.448.8600  
Fax: 800.888.7244, +(1) 760.448.8607  
USASales@HenryScheinOrtho.com  
IntlOrders@HenryScheinOrtho.com

Visit Henry Schein Orthodontics' website at:  
[HenryScheinOrtho.com](http://HenryScheinOrtho.com)

Symbols glossary provided electronically at  
[HenryScheinOrtho.com/symbols](http://HenryScheinOrtho.com/symbols)




INS135A  
Rev B

# K-Loop™ Distalizing Spring

**REF 034-000**

## INSTRUCTIONS FOR USE

Figure 1

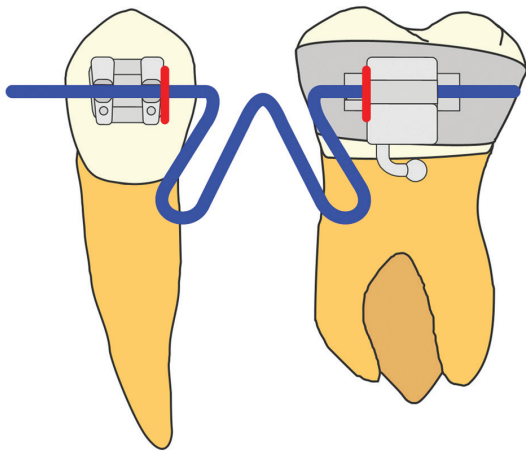

**PLACING THE K-LOOP—ACTIVATION**  
1. Insert the K-Loop into first molar tube and first premolar bracket. Place a mark just mesial to the molar tube and distal to the premolar bracket. (Figure 1)

The K-Loop is used for non-extraction cases to distalize molars in a more bodily fashion. While most distalizing appliances result in undesirable tipping of the molar, the special V-bend in the K-Loop moves both the crown and the root distally. The K-Loop is made of CNA™ Beta Titanium which produces gentle continuous forces for efficient and effective tooth movement.



2. Place 2mm high step bend 2mm distal to the molar mark and 2mm mesial to the premolar mark. (Figure 2)

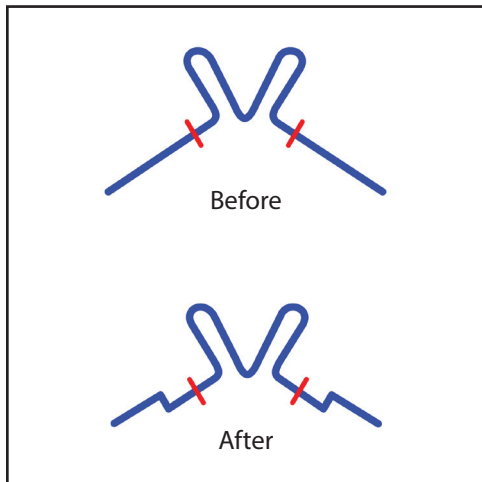


Figure 2

3. Insert K-Loop in place and ligate into premolar bracket. Place a cinch back bend mesial to the premolar bracket as shown. (Figure 3)

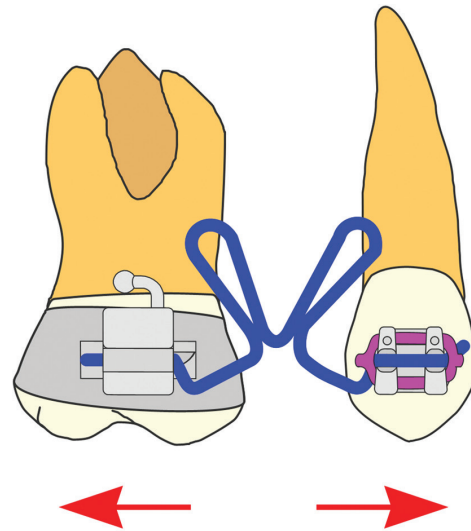


Figure 3

#### REACTIVATING THE K-LOOP (after 6-8 weeks)

NOTE: The K-Loop can be reactivated only once.

Figure 4

- Remove K-Loop from mouth
- Open the loop 1.5mm at "1"
- Open the loop 1.5mm at "2"
- Open the loop at "3" to gain original configuration
- Place again, following "Activation" instructions

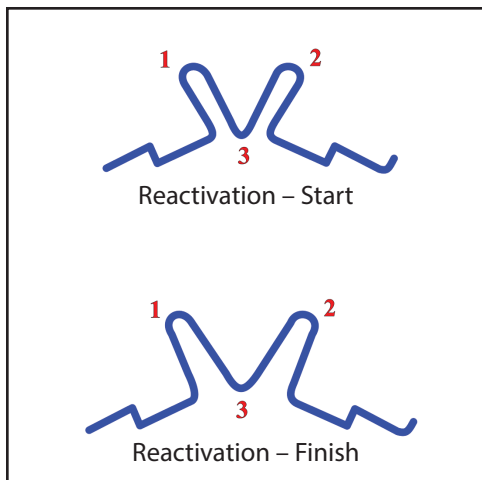


Figure 4

#### SUPPLEMENTAL INFORMATION

Kalra, V.: The K-Loop Molar Distalizing Appliance. *J. Clin. Orthod.* 29:298-301, 1995

Kalra, V.: Aparato de asa en K para distalizar molares. *J. Clin. Orthod. Edicion en Espanol*, 3:14-17, 1997

