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mdi Europa GmbH
Langenhagener Str. 71
30855 Langenhagen, Germany

(Single use only)

Printed in the United States of America.
U.S. patent #7,404,404

Rx Only

INS138A
Rev D

You, and your

LBB™
Lombardi
Bionator Bite Appliance

Not made with natural rubber latex.

REF 5200-100

(Catalog Number)

MASEL®

(Manufacturer)

Ortho Organizers, Inc.
1822 Aston Avenue • Carlsbad, CA 92008-7306
Ph: 800.547.2000 • 760.448.8600
Email: Sales@Masel.net
Website: MaselOrtho.com

LBB Patient Wear Guidelines

Congratulations! Your doctor has decided that you will benefit from wearing the LBB. Please follow these “How to Wear” instructions, so your treatment will progress as smoothly and quickly as possible.

Getting familiar with your LBB: Your LBB is made of a soft, comfortable material that allows it to fold over easily, and fit comfortably in your mouth as shown:

While not being worn, your LBB should be returned to its LBB case and stored flat. This recharges the hinges and keeps your LBB correctly positioned to last longer. An LBB that has been permanently misshaped should not be worn and needs to be replaced – call your doctor.

When you start using your appliance, you will notice that you will produce extra saliva because your body thinks the appliance is food. In the beginning, take extra slurps to swallow at first. In a few days, your body will get used to it and stop producing extra saliva, and you will find it much easier to swallow and sleep. Just be a patient with patience!

At first, your LBB may fall out during the night. That is okay. In a few days, you will get used to it and it will stop falling out. Be sure to wear it those awake hours!

When you should wear your LBB: Wearing your LBB as directed will help your treatment work better and faster. If your doctor limits your wear time, it probably means you are doing a great job and you can reduce the amount of time you need to wear it. Always follow your doctor's instructions if they differ from the following:

- Usually, the more you wear your LBB the better!
- Wear it for 3 to 4 hours during quiet daytime hours and all night long, 7 days a week.
- You should wear your appliance at home while awake during quiet time activities such as workouts, while reading, listening to music, watching television, while online or doing homework, and most importantly, all night long while you are sleeping.

Cleaning instructions: Clean daily with soap and water or a toothbrush with toothpaste. Rinse thoroughly.

In case of irritation: Smooth offending area with a nail file or emery board or carefully trim with a pair of scissors. If irritation continues, call your doctor.

If you become sick: You should soak the LBB and its case in a small pan with undiluted antiseptic mouthwash for 8 full hours to kill germs. Keep it flat while soaking. Rinse thoroughly with water before resuming wear. If you ever have strep throat, please consider calling your doctor to replace your LBB. You should also replace your toothbrush!

Keep your LBB away from pets! Dogs and cats will think it is food or a toy and will chew on it! When not being worn, your LBB should be stored flat in its case to protect it, recharge the hinges and keep it functioning effectively.

Don't lose your appliance! Doctors will charge you to replace it!

Remember: Bring your appliance to your appointments so that it can be checked regularly. If the LBB appliance becomes worn out or chewed up with bite marks or holes, ask your doctor if it should be replaced. That's better than doing that damage to your teeth!

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