Etch Gel

For Professional Use Only
Instructions for Use

INDICATIONS:

Etch Gel is indicated for initiation of the bonding sequence for bonding restorative materials to enamel and/or dentin.

DESCRIPTION:

Etch Gel contains phosphoric acid. It has a carefully balanced viscosity to achieve the following goals:

1. To allow easy dispensing.
2. To have the etch gel stay in the location where it is placed.
3. To allow the dispensed etch gel to quickly adapt to the dentin and enamel surfaces on which it is placed.

FORMULATION:

Etch Gel is a water based phosphoric acid gel.

DIRECTIONS FOR USE:

(If using a bonding resin system with instructions for phosphoric acid etching gels, the instructions included with that system should be used instead of the instructions below.)

1. The tooth should be cleaned by scaling and prophylaxis with flour of pumice. Isolate tooth of a rubber dam is highly recommended. Prepare any necessary cavity form according to normal dental procedures.
2. Apply etch gel to the tooth surface where etching of the enamel and/or dentin is required:
 - **Syringe delivery:** Remove the clear plug from the tip of the syringe. Small disposable bore metal syringe tips placed on the syringe allow dispensing of exact amounts of etch gel directly onto the surfaces to be etched. Place a syringe tip on the syringe and twist it into place until secure. Apply gentle pressure on the syringe plunger to dispense etch gel through the syringe tip directly onto the dentin or enamel to be bonded. CAUTION: Excess pressure may release an undesirable amount of etch gel. To avoid this, make sure there are no air bubbles in the syringe by dispensing a drop or two onto a mixing pad before beginning etching procedure. Discard the syringe tip after use and recap the syringe with the original cap to prevent drying of the etch gel.
 - **Bottle delivery:** Dispense 1 or 2 drops of etch gel into a mixing well or onto a pad. Apply the etch gel to enamel and/or dentin with a brush or sponge pellet. Recap the etch gel bottle immediately after dispensing.
3. Allow the etch gel to remain on enamel for 30 seconds. Agitation of the etch gel with soft brush bristles may enhance the etching process. Do not rub any hard material against the tooth during the etching process. Do not allow the etch gel to remain on dentin for more than 15 seconds. Carefully place the etch gel on the enamel for 15 seconds prior to additional application of the etch gel to the dentin.
4. Rinse the etch gel away. Start by using a few drops of water with high volume suction to avoid splashing the etch gel onto the tissues. Then wash with copious amounts of water using an oil-free air/water spray for 15 seconds.
5. Etched surfaces of enamel should have a dull, white, frosted appearance when etched and dried. If the enamel is not chalky appearing, repeat the etching process allowing etch gel to remain on the enamel for additional 30 second increments of time and wash and dry. Several repetitions of this etching procedure may be needed especially on primary teeth and teeth high in fluoride content.
6. Do not allow saliva contamination of the etched surface.
7. Follow instructions of the bonding agent or resin to be applied for management of the etched surface after the etching step.

WARNINGS:

- Dentists and assistants should wear gloves, masks and protective eye wear. Patients also should wear eye protection.
 - If contact with eyes occurs, flush immediately and for a prolonged period with copious amounts of water and consult a physician.
 - If contact with skin occurs, wash with soap and copious amounts of water.
 - In case of ingestion, seek medical attention immediately.
- Do not place etch gel directly on a pulp exposure.
- Avoid placing any acid etch gel on oral soft tissues.
- Acid etch gels are corrosive and can cause tissue damage in contact with skin and eyes.
- Do not use in individuals with allergies to these materials.
- Any non-sterilizable item that is handled in the delivery of the dental service should be disinfected by standard dental office hygiene procedures.
- Acid may be corrosive to metal surfaces. If any metal surface is contacted with the acid, wash the surface immediately.
- For additional information consult the Safety Data Sheet.

STORAGE:

- The system is designed to be stored between 50 °F – 75 °F (10 °C – 24 °C).
- Replace caps immediately after dispensing.
- Bring to room temperature prior to use.

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Etch Gel

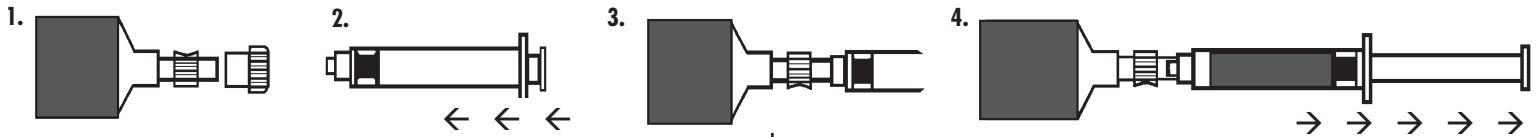
ADDITIONAL INSTRUCTIONS FOR 60 g SYRINGE KIT ONLY

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FILLING INSTRUCTIONS:

1. Remove cap from 60 g etch syringe.
2. Depress plunger of 3 cc empty delivery syringe to end of barrel.
3. Attach 3 cc empty delivery syringe to 60 g etch syringe through the luer lock coupling device of the 60 g etch syringe. Screw the 3 cc syringe onto the 60 g etch syringe by turning clock-wise. Turn only until tight.
4. Pull on plunger of 3 cc empty syringe to extract etch gel from the 60 g etch syringe. If necessary, gently push on plunger of the 60 g syringe.
5. To detach, retract plunger of 60 g etch syringe slightly to avoid leakage. Remove 3 cc delivery syringe by turning counter clockwise. Replace appropriate caps to both syringes. (Additional caps are provided for 3 cc syringes).

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EVEREST™
Orthodontic Adhesive

Manufactured for: (US Only)
Ortho Organizers, Inc.
1822 Aston Ave., Carlsbad, CA 92008
Ph: 760.448.8600/800.547.2000
HenryScheinOrtho.com

EC REP

mdi Europa GmbH

Langenhagener Str. 71
30855 Langenhagen, Germany