Manufactured for: (U.S. Only)
Ortho Organizers, Inc.

1822 Aston Avenue, Carlsbad, CA 92008-7306,

USATel: 800.547.2000, +(1) 760 448 8600

Fax: 800.888.7244, +(1) 760 448 8607

USASales@HenryScheinOrtho.com

IntlOrders@HenryScheinOrtho.com

mdi Europa GmbH

Langenhagener Str. 71

30855 Langenhagen, Germany

49 511 39 08 9530

WARNING: This product contains nickel and should not be used for individuals with known allergic sensitivity to nickel.

CAUTION: Federal law restricts this device to sale by or on the order of a licensed dentist or orthodontist.

For Single Use Only

Symbols glossary provided electronically at
HenryScheinOrtho.com/symbols
INS167A Rev C

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Genesis™

Active Self-Ligating Bracket System

Instructions For Use

1822 Aston Ave.

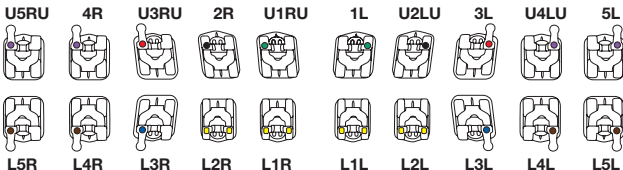
Carlsbad, CA 92008-7306 USA

In the U.S.: 800.547.2000

Customer Service: 888.851.0533

MaselOrtho.com

© 2017 Ortho Organizers, Inc.


Opening

To open the Genesis Active Self-Ligating Bracket spring clip, place instrument on the open clip slot directly beneath the bracket interface and gently push in the occlusal direction to disengage the bracket clip. Then place the instrument directly in the facial interface of the clip to engage the small extension of the spring clip. Pull in the occlusal direction. The spring clip will pop open.

Closing

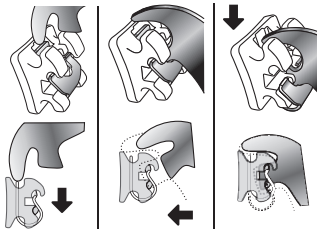To close the Genesis Active Self-Ligating Bracket spring clip, simply put light finger pressure on the occlusal surface of the bracket. Make sure the spring clip fully closes so the archwire is securely engaged

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1. Intended Use

The Orthodontic Bracket is a device intended for use in orthodontic treatment. The device is affixed to a tooth so that pressure can be exerted on the teeth.

2. Indication for use

Adhesive flash should be removed at bonding in order to minimize staining of the adhesive and to reduce the risk of bracket breakage during the debonding procedure. No special primers or pretreatments are necessary due to the bracket's special bonding base. Orthodontic brackets can be used with traditional direct or indirect bonding methods. For bonding procedures please follow the adhesive manufacturer's recommendations. When placing orthodontic brackets, it is suggested that the bracket be placed in a sliding motion, occlusal to gingival, forcing excess adhesive to the incisal edge.

3. Warnings

- To ensure proper function of the bracket tie-wires and avoid damage to bracket, do not strike the bracket with an instrument while attaching and removing elastomeric or metal ligature ties.
- Must dispose of and not reuse detached product.
"SINGLE USE ONLY"
- Must follow orthodontist's direction if using for a different application.
- Instruct patients not to chew or bite on hard substances such as hard candy, ice, carrots, etc.
- If an allergic reaction occurs, direct patient to consult their dentist.
- It is recommended to bond these orthodontic brackets using a light curing adhesive.

4. Contraindications

- Heavy smoking, tobacco and alcohol abuse.
- Systemic blood disorders.
- Uncontrolled diabetes.

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