Distributed by:

Ortho Organizers, Inc.
1822 Aston Avenue
Carlsbad, CA 92008
USA
Tel: 800.547.2000, +(1) 760 448 8600
Fax: 800.888.7244, +(1) 760 448 8607
USASales@OrthoOrganizers.com
IntlOrders@OrthoOrganizers.com

Visit the *Ortho Organizers'* Web site at
OrthoOrganizers.com

mdi Europa GmbH
Langenhagener Str. 71
30855 Langenhagen, Germany
49 511 39 08 9530

OMA Member
Orthodontic Manufacturers
Association

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Porcelain Conditioner REF 480-013
INSTRUCTIONS FOR USE

Porcelain Conditioner prepares dental porcelain for metal or ceramic direct bond brackets. If needed, porcelain conditioner will improve the bonding of ceramic bracket to enamel. This conditioner is a silane coupling agent which helps to chemically bond the porcelain to the bonding adhesive. (If you are applying primer to a ceramic bracket that will be bonded to enamel, omit steps 1 and 2 below.)

1. The porcelain bonding surface should be pumiced and dried.
2. An etching solution should be applied to the porcelain for 60 to 90 seconds.
3. Porcelain conditioner should then be applied to the clean porcelain crown or ceramic bracket with a cotton pellet. Leave conditioner on for 1 minute. The surface should then be rinsed and dried.
4. The porcelain is now ready for bonding. Ideal bonding occurs on a smooth, glazed surface. If bonding fails after following the above directions, it is necessary to break the glaze with a stone and repeat the procedure.
5. Removal of bonded brackets requires care, as porcelain is a brittle substance. Removal is best accomplished using a peel-type force. Squeezing the bracket wings toward each other will usually separate the bracket from the adhesive. Any remaining adhesive can be removed with a rubber wheel.

CAUTION: Porcelain conditioner is flammable and should be used carefully. Avoid contact with eyes and soft tissue.

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