## INSTRUCTIONS FOR USE

### Description:

*Illuminate™* Light-Cure Adhesive is a single paste resin-based, fluoride containing bracket adhesive. The single paste adhesive requires no mixing; resulting in preferable handling properties, reduced adhesive waste, consistent adhesive performance, and no working time constraints.

### Indications for Use:

*Illuminate* Light-Cure Adhesive is an orthodontic adhesive designed to bond metal, ceramic, and plastic brackets to enamel.

### Contraindications:

*Illuminate* Light-Cure Adhesive is contraindicated in patients with a history of severe allergic reaction to methacrylate resins.

### General Precautions:

- Do not use in the presence of eugenol based materials as complete setting may not occur.
- Metal, Porcelain Crowns or Facings: Do not bond to crowns that have thin facings or that appear to be compromised. Prepare the surface of the crown or facing to be bonded by using the appropriate conditioner. Follow the instructions included with the conditioner.
- Do not expose to elevated temperatures or intense light.

### Adverse Reaction:

Allergic or pre-existing skin, eye, and respiratory conditions may be aggravated by exposure to this product.

### Instructions for Use:

#### Tooth Preparation

1. Thoroughly clean teeth to be bonded with No Added Color/Flavor/Fluoride Prophylaxis Paste. Prophylaxis paste which contains oil or fluoride should not be used as it may leave a residue which can inhibit bonding.
2. Rinse teeth thoroughly with water; dry with oil and moisture-free compressed air.
3. If using *Illuminate* Etching Gel, etch enamel for 15 seconds; never leave etchant on tooth for more than 60 seconds. Lightly agitating the etchant during its application can enhance its action.
4. Rinse teeth thoroughly with water to remove all traces of the etchant. Re-isolate and dry with oil-free and moisture-free compressed air. The surface should appear dull and frosty white. If not, repeat step 3.

#### Bracket Adhesive Application

1. Dispense two drops of *Illuminate* Light-Cure Bonding Resin onto a dispensing pad or other suitable surface. With a brush apply one thin uniform layer onto each etched and dried tooth surface to which the bracket is to be bonded. It is not necessary to cure the *Illuminate* Light-Cure Bonding Resin at this time. However, to minimize floatation and contamination, the *Illuminate* Light-Cure Bonding Resin may be cured for 10-15 seconds. If contamination has occurred to the cured *Illuminate* Light-Cure Bonding Resin surface it can be easily cleaned by dabbing the cured surface with alcohol without the need to re-etch. To enhance bond strength with some fine mesh brackets apply a thin uniform layer of *Illuminate* Light-Cure Bonding Resin to the bracket base and cure for 10-15 seconds before applying paste.
2. Apply a thin layer of *Illuminate* Light-Cure Adhesive to the bracket base. Place the bracket onto the tooth and press lightly in the desired position. Excess adhesive can be removed easily from the perimeter of the bracket base with an explorer or plastic instrument after bracket is in place.
3. With a metal bracket, using a light-curing unit (minimum output of 600mW/cm<sup>2</sup>) shine the light-curing unit tip from the incisal edge and illuminate for 20 seconds and for 15 seconds from the gingival, mesial or distal bracket edge. Curing may be done directly through transparent brackets from the labial for 20 seconds. For all bracket materials, the light-curing unit tip must be placed as close to the bracket base as possible during curing.
4. Archwires can be placed immediately once the paste has been properly cured.
5. After treatment is complete, remove brackets in the usual manner.

#### Storage:

- Recommended storage conditions are between 2°C/36°F to 27°C/80°F, bringing to room temperature before using (22°C/72°F). Do not freeze. See outer package for expiry date.
- Do not expose to elevated temperatures or intense light.
- Keep containers closed when not in use.

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Ortho Organizers, Inc.
**mdi Europa GmbH**