Manufactured for:

Ortho Organizers, Inc.
1822 Aston Avenue
Carlsbad, CA 92008-7306
USA
Tel: 800.547.2000, +(1) 760 448 8600
Fax: 800.888.7244, +(1) 760 448 8607
USASales@HenryScheinOrtho.com
IntlOrders@HenryScheinOrtho.com

Visit the *Henry Schein Orthodontics* Web site at
HenryScheinOrtho.com

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Tooth Preparation

- Prepare teeth with a non-fluoride, oil-free prophyl paste. Flush thoroughly with water.
- Isolate teeth with cotton rolls or rubber dam.
- Air dry completely with an oil/moisture-free air source.

Etching Surfaces

- Apply one or two drops of etchant onto pad. Using a cotton pellet or brush, dab etching agent onto teeth to be bonded.
- Allow to remain for 60 seconds. Try to avoid contact with gingival tissue.
- Thoroughly rinse the etching agent from the enamel surface with excessive amounts of water to remove all traces of acid (do not wipe etching agent off the enamel surface).
- Air dry thoroughly using oil-and-moisture-free air source. Teeth should have a dull, white frosted appearance.

Priming Surfaces

- Apply a thin uniform coating of primer onto each tooth surface to be bonded.
- Next, apply a thin coating of primer to the underside of each bracket base.

Adhesive Bonding

- Apply a small amount of adhesive paste onto the bracket base and place on tooth. In 30 seconds, position the bracket to desired angulation and press firmly.
- Excess adhesive from around the bracket base can be easily removed after all brackets are in place.
- Archwires can be placed 5 minutes after bonding of the last bracket.

Indirect Bonding

- Prophyl and etch tooth as prescribed. Place adhesive in a syringe, prime all the teeth to be bonded and all bracket bases in the tray. Using the syringe, dispense a small amount of paste onto each primed bracket base in the tray. Place tray in mouth, wait three minutes and peel tray material away from tooth surface.

Storage and Use

- This system is designed to be used at room temperature of approximately 21° to 24°C or 70° to 75°F. Shelf life at room temperature is one year for unopened kits and 90 days for opened kits when caps from jars, bottles, syringes and tips are replaced immediately after dispensing.

Note: Although Advantage is hypo-allergenic, prolonged skin contact should be avoided.

Warning: Etchant contains a 37% Phosphoric Acid solution which can be harmful to skin or eyes. In case of contact, flush with water and seek medical assistance for appropriate treatment.

Advantage No-Mix Direct Bonding Adhesive
REF 480-000, 480-000S, 480-000SP, 480-001 480-020
 GEBRAUCHSANWEISUNG

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