Manufactured for: (US Only)

Ortho Organizers, Inc.

1822 Aston Avenue

Carlsbad, CA 92008-7306

USA

Tel: 800.547.2000, +(1) 760 448 8600

Fax: 800.888.7244, +(1) 760 448 8607

USASales@OrthoOrganizers.com

IntlOrders@OrthoOrganizers.com

Visit the *Ortho Organizers'* Web site at  
**OrthoOrganizers.com**

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**REF 600-150**

INSTRUCTIONS FOR USE

1. Store in Ambient temperature of 55 to 75° F.

2. Seal container after using.

**CAUTION**

Do not take internally. Avoid contact of paste with eyes or skin. Use only in well ventilated areas or with ventilation equipment. If contact with eyes, flush immediately with clear water for 15 minutes. CONTACT A PHYSICIAN.

**WARNING**

Protect yourself and others. Read and understand this label. FUMES & GASES can be dangerous to your health. HEAT RAYS (INFRARED RADIATION from flame or hot metal) can injure eyes. Read and understand the manufacturer's instructions and your employer's safety practices. Wear correct eye, ear, and body protection.

**CAUTION: CONTAINS FLUORIDES!**

This paste, when heated, gives off fumes that may irritate eyes, nose, and throat. Avoid fumes—use only in well ventilated spaces. Avoid contact of paste with eyes or skin. Do not take internally. Keep out of reach of children. See American National Std. Z49.1 Safety in Welding and Cutting, published by American Welding Society, 550 North Lajune Rd., Miami, FL 33135 OSHA Safety & Health Standard, 29CFR1910, available from the US Government Printing Office Washington, DC 20402.

**WARNING**

This material has been manufactured to its standard specifications. Defects must be reported to Ortho Organizers, Inc. within 90 days of manufacture. Liability is limited to replacement of defective material.

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