Distributed by:

Ortho Organizers, Inc.
1822 Aston Avenue
Carlsbad, CA 92008
USA

Tel: 800.547.2000, +(1) 760 448 8600
Fax: 800.888.7244, +(1) 760 448 8607
USASales@OrthoOrganizers.com
IntlOrders@OrthoOrganizers.com

Visit the *Ortho Organizers'* Web site at
OrthoOrganizers.com

mdi Europa GmbH

Langenhagener Str. 71
30855 Langenhagen, Germany
49 511 39 08 9530

OMA Member
Orthodontic Manufacturers
Association

**Made
in the
USA**

INS29A
Rev C

Utility Arch

**REF 100-273 to -280, -283 to -285,
-314, -334, -338 and -342,
105-019 and 105-019S**

INSTRUCTIONS FOR USE

Step 2
Lengthen archwire—if necessary—by grasping the square outer tubes of distal utility arch, while simultaneously pulling the anterior portion forward to approximate length.

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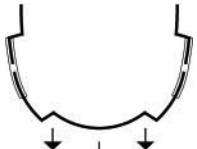
Step 3

Place distal ends of arch in upper or lower 1st molar buccal tubes. Make sure the midline is placed between the central incisor brackets and engage—but do not ligate.

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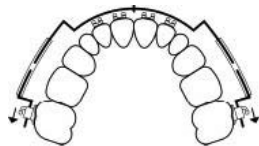
FIGURE C

Place into anterior brackets and buccal tubes. Check midline.

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Step 6

For best results intraorally, use an Angled Utility Arch Plier (Ortho Organizers' # 201-437), to crimp mesial portion of the outer tube. This will secure the wire at the desired length and prevent it from sliding into the tube.

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Step 4

The 3mm step in the archwire should be flush with the mesial part of the buccal tube and should tie toward the gingiva.

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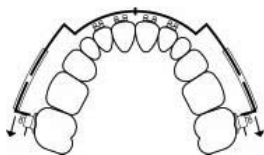
FIGURE D

Slide wire against mesial of buccal tube.

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Step 7

Plier placement for molar rotation.

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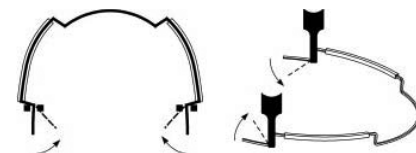
FIGURE G

Step 5

Make any further positioning adjustments to the archwire at this time before crimping. (See Steps 7 and 8 for other possible adjustments.)

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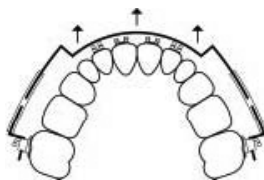
FIGURE E

Advance anterior bridge 1mm for incisor advancement if needed.

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