**Ortho Organizers, Inc.**  
1822 Aston Avenue  
Carlsbad, CA 92008-7306  
USA  
Tel: 800.547.2000, +(1) 760.448.8600  
Fax: 800.888.7244, +(1) 760.448.8607  
USASales@HenryScheinOrtho.com  
IntlOrders@HenryScheinOrtho.com

Visit the Henry Schein Orthodontics website at [HenryScheinOrtho.com](http://HenryScheinOrtho.com)

**mdi Europa GmbH**  
Langenhagener Str. 71  
30855 Langenhagen, Germany  
49 511 39 08 9530

Symbols glossary provided electronically at [HenryScheinOrtho.com/symbols](http://HenryScheinOrtho.com/symbols)

**Made in the USA**

INS29A  
Rev D

English \_\_\_\_\_  
**Contraindications**  
This product contains nickel and chromium and should not be used for individuals with known allergic sensitivity to these metals.

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
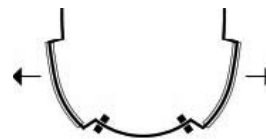
**Step 2**  
Lengthen archwire—if necessary—by grasping the square outer tubes of distal utility arch, while simultaneously pulling the anterior portion forward to approximate length.

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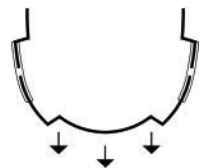### Step 3

Place distal ends of arch in upper or lower 1st molar buccal tubes. Make sure the midline is placed between the central incisor brackets and engage—but do not ligate.

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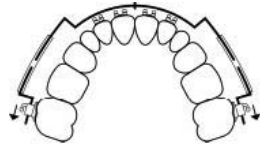
FIGURE C

Place into anterior brackets and buccal tubes. Check midline.

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### Step 6

For best results intraorally, use an Angled Utility Arch Plier (Ortho Organizers' # 201-437), to crimp mesial portion of the outer tube. This will secure the wire at the desired length and prevent it from sliding into the tube.

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### Step 4

The 3mm step in the archwire should be flush with the mesial part of the buccal tube and should tie toward the gingiva.

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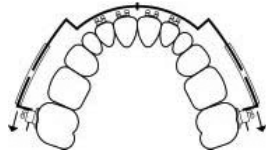
FIGURE D

Slide wire against mesial of buccal tube.

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### Step 7

Plier placement for molar rotation.

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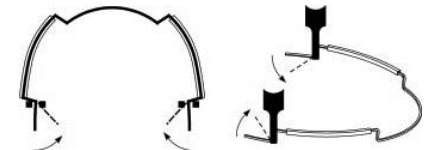
FIGURE G

### Step 5

Make any further positioning adjustments to the archwire at this time before crimping. (See Steps 7 and 8 for other possible adjustments.)

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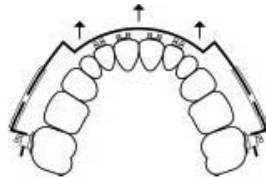
FIGURE E

Advance anterior bridge 1mm for incisor advancement if needed.

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