MAGNET FORCE SYSTEM™

REF 424-200

INSTRUCTIONS FOR USE, MODE D'EMPLOI, GEBRAUCHSANWEISUNG, ISTRUZIONI PER L'USO, INSTRUCCIONES DE USO

Standard Method, Méthode standard, Standardmethode, Metodo standard, Método estándar

1. Correct alignment of the molar buccal tubes is important in order to simplify magnet insertion. To align, use a long wire inserted into the molar buccal tube during band cementation and extend the wire mesially and parallel to the buccal surface of the 1st and 2nd premolars.

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2. The long wire shown in Fig.1 should also be parallel to the occlusal as shown in Fig. 2.

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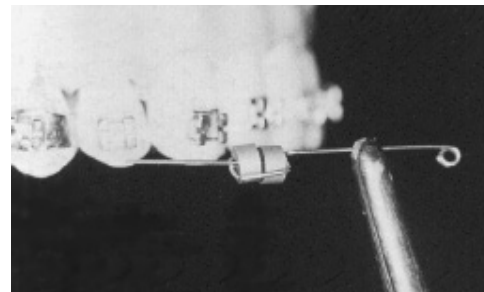
3. For anchorage, use a modified Nance which should be connected to the 2nd premolar bands and then cemented. The acrylic Nance button should be located in the area of the greatest curvature of the palatal vault and should extend laterally and anteriorly for maximum anchorage. **NOTE:** Brackets shown in Fig. 3 are used when required, although brackets anterior to the second premolar are not essential in the Standard Method of Insertion (which provides a distinct esthetic advantage during the difficult first phase of molar distalization).

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
4. Shown above is one of the two sectional .016 stainless steel wires (with magnets on the wire) included in each kit. The 2 magnets are repelling each other and are held together by a restraining wire clamp attached to a bent stop. They have been prepared for insertion directly into the most buccally positioned tube. **NOTE:** the mesial helix on the wire will be used to ligate (and therefore activate) the magnets when the restraining wire clamp is later removed.

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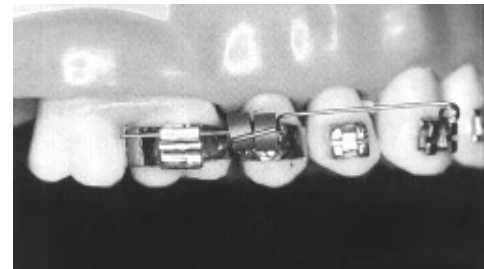5. Insert the distal end of the .016 sectional wire into the most buccally positioned Edgewise tube.

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6. The amount of distalization desired will determine the length of the wire from the bent stop to a mesial helix which will need to be newly formed by the Doctor. For example, 4mm of distalization would require about 4mm distance from the bent stop to the newly formed helix. The band on the 2nd premolar may or may not have a bracket but always requires a welded short vertical tube deep in the distobuccal embrasure for ligature activation from the mesial helix.

Further enhancement fo the Nance anchorage may be accomplished by soldering a lingual arch to the Nance from one 2nd premolar to the contralateral 2nd premolar. Always remember to activate the magnets by ligating from the shortened mesial helix to the vertical tube on the 2nd premolar band while

pushing the magnets distally with your finger and be sure to remove the restraining clamp. **NOTE:** The distal end of the .016 wire should be allowed to extend about 3mm distal to the buccal tube and should then be bent lingually to avoid buccal irritation.

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
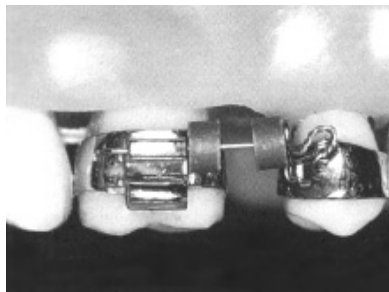
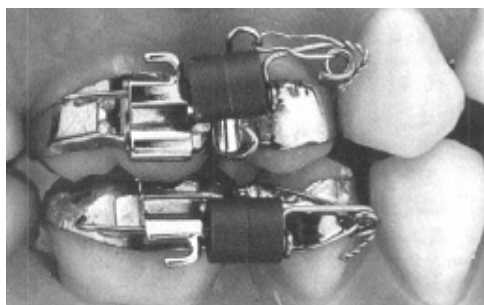
9. Magnets used to simultaneously distalize maxillary and mandibular molars without occlusal interferences.

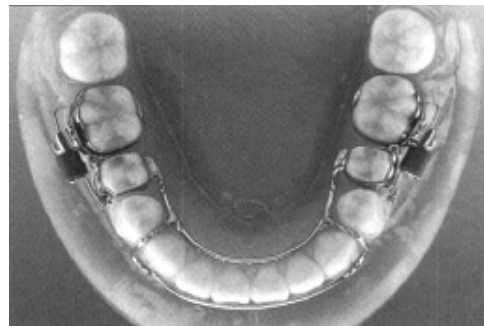Aimants utilisés pour distalizer simultanément les molaires supérieures et inférieures sans interférence occlusale.
Zur gleichzeitigen Distalisierung maxillärer und mandibulärer Molaren ohne okklusale Störungen verwendete Magneten.
Calamite utilizzate per distalizzare contemporaneamente i molari mascellari e mandibolari senza interferenze occlusali.
Imanes usados para distalizar simultáneamente los molares maxilares y mandibulares sin interferencias oclusales.

10. Mandibular lingual arch anchorage from 2nd premolar to contralateral 2nd premolar. A modified Hawley extension from the lingual arch may also be used to enhance mandibular anchorage. This Hawley extension can further augment anchorage by providing reverse torque.

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11. Bilateral magnets in the maxillary arch with Nance and lingual arch anchorage.

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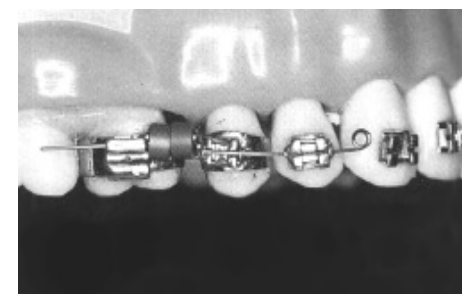
You may now elect to use either the Standard Method of insertion or the Alternative Method. The Standard Method is simpler and has esthetic advantages while the Alternative Method provides better control of distal bodily movement.

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12. Steps 1 through 5 remain the same as in the Standard Method but, unlike the Standard, there is no need to shorten the mesial helix. Also, NOTE the presence of brackets mesial to the 2nd premolar. Push the .016 sectional wire distally until the magnet touches the buccal tube. Bend the distal end of the wire lingually to avoid buccal irritation, about 3mm distal to the buccal tube. The sectional wire should be ligated to the 1st and 2nd premolar brackets. Ligate the activating ligature from the helix at the mesial end of the wire to the 2nd premolar bracket to activate the magnets while keeping them in maximum contact. Remember to remove the restraining clamp.

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13. While tightening the activating ligature, it is important to push both magnets distally with your finger and hold them in order to achieve maximum contact. Test to be sure that magnets remain together in contact when your finger is removed.

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Distributed by:

ORTHO
Organizers
A HENRY SCHEIN COMPANY

Ortho Organizers, Inc.
1822 Aston Avenue
Carlsbad, CA 92008
USA

Tel: 800.547.2000, +(1) 760 448 8600
Fax: 800.888.7244, +(1) 760 448 8607
USASales@OrthoOrganizers.com
IntlOrders@OrthoOrganizers.com

Visit the Ortho Organizers' Web site at OrthoOrganizers.com

mdi Europa GmbH
Langenhagener Str. 71
30855 Langenhagen, Germany
49 511 39 08 9530

OMA Member
Orthodontic Manufacturers
Association

CE
0470

Made
in the
USA

INS34B
Rev A