INSTRUCTIONS FOR USE

TransForce^{®2} Transverse Arch Developing Appliances

REF 424-500, 424-526 – 424-532

TransForce^{®2} Transverse Arch Developing Appliances are manufactured by Ortho Organizers[®] as a series of pre-adjusted fixed appliances designed to correct arch form for patients with contracted dental arches. The appliances deliver smooth continuous force with a pre-activated expansion module that incorporates a nickel titanium coil spring enclosed in a tube.

The TransForce^{®2} Transverse Arch Developing Appliance is designed to increase the inter-canine width and may be used in upper or lower arches where expansion is required to accommodate crowding in the labial segments, or to correct arch width in contracted arches. The expansion module of the Transverse Appliance is calibrated to approximately 200 grams of force, in accordance with the force requirements needed in arch development for transverse expansion. No activation is required after fitting the appliance. As a fixed appliance, TransForce^{®2} Transverse Arch Developing Appliances eliminate the challenges with non-compliant patients.

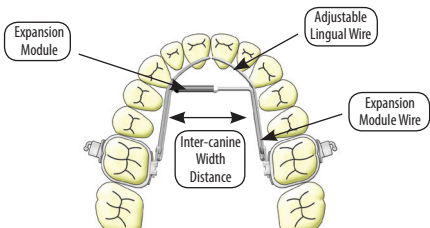
Figure 1—Schematics of TransForce^{®2} Transverse Appliance

Contraindications

This product contains nickel and chromium and should not be used for individuals with known allergic sensitivity to these metals.

Arch Development Planner

An Arch Development Planner is provided in each kit to assist in selecting the correct size. This clear template shows each appliance to scale in both fully compressed and fully expanded forms. A millimeter scale is provided on the template to help you measure.

Selecting the Correct Size

1) Measure the inter-canine width and inter-molar width on study model by measuring from the gingival margin of the right cuspid to the gingival margin of the left cuspid and the gingival margin of the right molar to the gingival margin of the left molar.

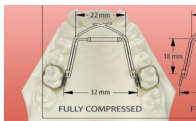
Figure 2

2) Place planner over the patient's study model to find the size to fit your selected case. Find the FULLY COMPRESSED appliance size that best fits inside the lingual outline of the teeth (Figure 2). Compare to the FULLY EXPANDED image to see the total amount of expansion capable by that appliance.

You can achieve up to 8 mm of expansion per TransForce^{®2} Transverse Appliance. In cases that need more than 8 mm of expansion, a second appliance can be used to achieve desired expansion.

Appliances are color-coded for easy identification (Table 1).

Catalog No.	Color-Code	Inter-canine Fully Compressed	Inter-canine Fully Expanded	Maximum Amount of Expansion
424-526	Red	18 mm	26 mm	8 mm
424-528	Green	20 mm	28 mm	8 mm
424-530	Purple	22 mm	30 mm	8 mm
424-532	Pink	24 mm	32 mm	8 mm

Table 1—Measurement for TransForce^{®2} Transverse Appliance

3) At any stage in treatment, you can de-activate the appliance by crimping the expansion module tube to compress it on the wire and prevent any further expansion.

Appliance Fitting

1) Within three days prior to placing the appliance, prepare the patient for molar bands by placing separators.

Figure 3

Inserting the Appliance in the Lingual Sheath

2) On the day of placement, select molar bands with lingual sheaths and confirm correct size of bands.

3) Insert the appliance into both lingual sheath assemblies until the distal extension blade snaps into the latching indent of each sheath (Figure 3). The lingual wires should be towards the occlusal edge and the expansion module wire should be towards the gingival. Weingart pliers (Catalog No. 205-202) can be used to insert the placement blade into sheath.

Figure 4

Securing the Appliance in the Lingual Sheath Using Elastic Ligature

4) For additional support, tie in the appliance by using an elastic ligature from the front mesial hook of the blade around to the back of the blade (Figure 4).

5) The appliance should be tried on patient's model prior to cementing to verify size and to make any adjustments where necessary (Figure 5).

Figure 5

Fitting the Appliance Prior to Cementing

Note: It is easier to attach the molar bands to the TransForce^{®2} Transverse Appliance and fit the appliance in as one piece, rather than fitting the bands first then inserting the appliance in the lingual sheaths.

6) Minor adjustments may be required to adapt the appliance to each individual case:

Lingual Wires Anterior Section

The lingual wires are adjustable and may be used to align irregular anterior teeth from the lingual aspect. The lingual wires should be near the gingival margin of the teeth and can also be trimmed if shorter arms are preferred.

Note: It is essential to expand the inter-canine width to accommodate lingually displaced incisors. Therefore, the lingual wires should remain passive until the necessary space has been created.

Expansion Module Wire Anterior Section

If the appliance is slightly too big and does not clear the bicuspid, you can adjust the appliance by placing a bend on the anterior corners of the expansion module wire using pliers.

Expansion Module Wire—Lingual Sheath Section

You can bend the expansion module wire towards the occlusal margin to prevent tissue impingement. If less expansion of molar width is required, the expansion module wire may be held with pliers distal to the canines and bent lingually. Alternatively if additional expansion of molar width is required, the expansion module wire may be held with pliers distal to the canines and bent buccally.

Figure 6

Using an Elastic to Hold the Appliance in Compressed Position

7) Compress the appliance to enable the lingual wire to fit within the dental arch and maintain compressed position with a ¼, 6 ½ oz. elastic (Figure 6).

8) Mix band cement and place in both molar bands.

9) Place and seat one molar band and then the other band. The expansion module should be positioned toward the gingival. Check for secure fit until the cement has hardened. Clean off excess cement after cement has set and remove the elastic band (Step 7) to activate the appliance.

10) The device is activated immediately and will provide slow, continuous force to expand the arch up to 2 mm every 6 to 8 weeks. The appliance can achieve up to 8 mm of total expansion.

11) After placement and throughout treatment, additional minor adjustments can be made to the appliance intra-orally, if necessary.

Maintenance

After fitting the appliance, maintenance should be minimal and may be limited to routine visits of 6- to 8-week intervals to check progress.

If lingual wires are used to control the alignment of the incisors, a slight adjustment may be necessary. This is normally done intra-orally with triple-beak or concave-convex pliers.

Retention / Removal

The appliance can be de-activated and left in place as a retainer after the necessary expansion has been accomplished. To de-activate the appliance crimp expansion module tube onto the wire.

To remove the appliance, simply stretch and remove elastic ligature, then snap the appliance out of each molar lingual sheath with Weingart pliers (Catalog No. 205-202).

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Ortho Organizers and TransForce are registered trademarks of Ortho Organizers, Inc.
Ortho Organizers, Inc.
mdi Europa GmbH